

DISGARD

the anti-needlestick sharps pad

The DisGard sharps pad allows the safe collection and disposal of used blades, suture needles and other sharps. DisGard is a robust, clear plastic hinged wallet with an adhesive pad to hold the sharps during clinical procedures. The product has an adhesive backing for stabilisation at the clinical site.



- clear visual control at all times
- enhances audit and control procedures
- strong adhesive pad, retains sharps during the procedure
- sterile single use device
- resistant to needle or blade puncture
- low cost safety solution
- retains fluid on closure

SAFE**GARD**
Medical

DISGARD

the anti-needlestick sharps pad

instructions for use:


- 1** Secure DisGard to surface, such as the patient's drape by removing the small peel off patch on the base to reveal an adhesive pad. Apply firm downward pressure to affix DisGard to the surface.



- 2** Remove paper strip covering the foam pad to reveal adhesive surface.




- 3** Place discarded sharps onto the adhesive area of the pad using the appropriate instrument.

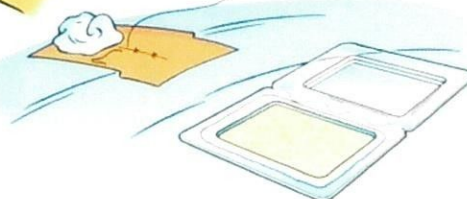

- 4** Place discarded sharps onto the yellow area of the pad using the appropriate instrument.



- 5** The procedure complete, fold lid over at the hinge and seal. The sharps remain visible for counting.

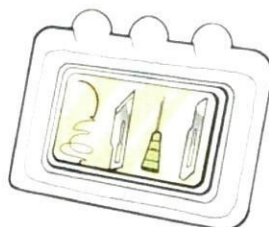

- 6** Dispose of the DisGard in the appropriate manner.



Manufactured by

Safegard Medical (Hungary) Kft.
3608 Farkaslyuk,
Jószerecsét tér 15.
HUNGARY

Tel.: +36-48/ 470-758

Fax: +36-48/ 474-798

e-mail: sales@safegardmedical.com

www.safegardmedical.com