

Instructions for Use with Attached Needle

- 1 **Peel Package** as indicated, to allow proper removal of the **SECUREGARD** Retractable Safety Syringe and remove needle cap.
- 2 Prepare vial for medication draw up. **Do not push the syringe plunger past the first gradation mark.**
- 3 **Draw up medication** per facility protocol. (See Figure 1).
- 4 Prepare injection site and administer medication per facility protocol.
- 5 **To engage the safety mechanism, depress the plunger FULLY** to enable retraction of the needle. (See Figure 2).
- 6 **Pull back on plunger until it stops:** observe withdrawal of needle into barrel. (See Figure 3).
- 7 **Snap off plunger** to permanently disable syringe. (See Figure 4).
- 8 **Discard immediately in sharps container** per facility protocol.



Figure 1



Figure 2



Figure 3



Figure 4

Caution

The **SECUREGARD** Retractable Safety Syringe **should not be used** to take blood samples.

This device is sterile unless the package has been damaged or opened. Once the needle has been retracted into the barrel, **do not depress the plunger**. If unable to activate safety mechanism, discard syringe immediately into an approved sharps container.

This device is a single use item only.

