

INSTRUCTIONS FOR USE:

1. To open package, peel tabs apart to remove syringe. Ease needle sheath off.
2. Prepare medication vial. Do not push the syringe plunger past the first gradation mark because the safety mechanism is activating rendering the syringe unusable.
3. Draw medication per facility protocol. (See Figure 1)
4. If necessary, allow air into the ampoule but be careful not to push the syringe plunger past the first gradation mark.
5. Prepare injection site and administer medication by FULLY pressing the plunger. (See Figure 2)
6. Pull back the plunger until it stops: observe withdrawal of needle into barrel. (See Figure 3)
7. Snap off plunger to permanently disable syringe. The needle sheath and the broken part of the plunger can be recycled. (See Figure 4)



1. figure



2. figure














3. figure



4. figure

The meanings of the symbols shown on the labels:

STERILE	EO	Sterilize using ethylene oxide		Consult instructions for use
		Do not use if package is damaged		Caution
		Re-use prevention		Non-pyrogenic
		Do not re-use		Latex free
		Do not re-sterilize		Temperature limit
REF		Catalogue number		Manufacturer
LOT		Batch code		Use by date

Caution

SECUREGARD syringes are single use devices only. Once the needle is fully retracted, do not press the plunger again. If unable to activate safety mechanism, discard syringe immediately into an approved sharps container. This device should not be used to take blood samples. The device is sterile unless the package has been damaged or opened. Do not use the device after the expiration date.

Warning

Any re-use or re-sterilization could create the risk of cross-contamination of the device, which could result in patient infection or cross-infection.